

INTRODUCING

ErbaQik[®] COVID-19 Ag Test

One step rapid immunochromatographic assay for the detection of SARS-CoV-2 virus in human nasopharyngeal sample.

Approved by



icmr
INDIAN COUNCIL OF
MEDICAL RESEARCH
Serving the nation since 1950



Ideal for early detection of
COVID-19 suspects



Detect all COVID-19 variants including
OMICRON, Delta + and Deltacron



Best suited for point of care testing
in remote locations, quickly



Easy to use test, requiring
minimal skills

INTRODUCING

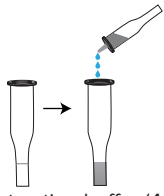
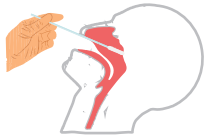
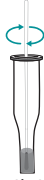
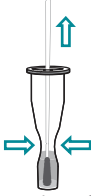

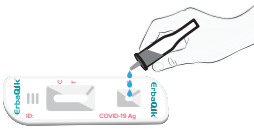

ErbaQik COVID-19 Ag Test

ErbaQik COVID-19 Ag test is an easy to use rapid antigen test which is intended for professional use as an aid in diagnosis of COVID-19 which is caused by SARS-CoV-2 virus.

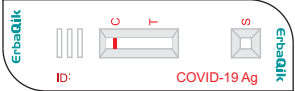
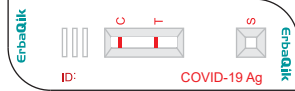
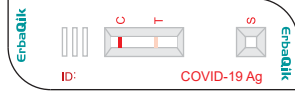
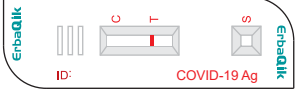
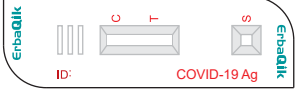
ErbaQik Covid 19 Ag: Test you can depend on

- Uses highly purified SARS-CoV-2 nucleocapsid Ab
- Easy sample collection
- Results in just 15 minutes
- Easy result interpretation
- Highly accurate
- Approved by ICMR

Assay procedure

Step 01  Fill the Extraction buffer (400 µl)	Step 02  Nasopharyngeal swab collection	Step 03  Insert and stir the swab in extraction buffer 8-10 times	Step 04  Squeeze the sides of the Tube
Step 05  Place the nozzle	Step 06  Add 3 Drops (100 µl) of solution into the sample well	Step 07  Read the results at 15 minutes	

Interpretation of the result

Negative	Positive	Invalid
 One band Control "C" in the result window	 One band "T" and One band in "C" in the result window  One band "T" and One band in "C" in the result window	 No Control band "C" in the result window 

Ordering information

Product Code	Product Name	Pack Size	Shelf-life
134335	ErbaQik COVID-19 Ag Test	25 Tests	24 months

In Collaboration with

ERBA Diagnostics Mannheim GmbH

Mallaustrasse, 69-17, 68219, Mannheim, Germany

Tel: +49 621 8799770 Fax: +49 8799688

E-mail: sales@erbamannheim.com

www.erbamannheim.com

Transasia Bio-Medicals Ltd.

Transasia House, 8, Chandivali Studio Road,
Chandivali, Andheri (E), Mumbai - 400 072

T: (022) 4030 9000 F: (022) 2857 3030

✉ responses@transasia.co.in

